

## Step 1

Place the mask on your face with the nosepiece positioned on top and the attached side of the elastic band facing inward, toward the mouth.



## Step 2

Stretch the elastic bands on either side away from the mask and secure them behind your ears.



## Step 3

Position the nosepiece so it is centered directly over your nose, and then bend it on both sides for a snug fit around your nose.



## Step 4

Pull the lower part of the mask down and extend the pleats so the mask fully covers your nose to your chin.



### To Order:

- Call 800-975-0563
- Scan the QR Code on the right
- Email [Info@covenant-hcec.com](mailto:Info@covenant-hcec.com)
- Visit [www.covenant-hcec.com](http://www.covenant-hcec.com)

Exclusively Distributed by:

**COVENANT**  
HEALTHCARE EQUIPMENT & CONSULTING



## Precautions for Use

- This product is a disposable mask. Do not wash and reuse.
- If you feel abnormal after wearing this product, stop using it immediately.
- This product is not an industrial dust and gas mask. Do not use the mask in environments generating harmful gas or dust.
- Be careful not to hurt your face or hands with the nosepiece.
- To maintain proper hygiene, replace the mask immediately when it gets dirty.

## Storage

Store at room temperature in a clean, dry place, away from direct sunlight. Keep the product out of reach of children.



## To Order:

- Call 800-975-0563
- Scan the QR Code on the right
- Email [Info@covenant-hcec.com](mailto:Info@covenant-hcec.com)
- Visit [www.covenant-hcec.com](http://www.covenant-hcec.com)

Exclusively Distributed by:

**COVENANT**  
HEALTHCARE EQUIPMENT & CONSULTING

