



Step 1

Separate mask to open.

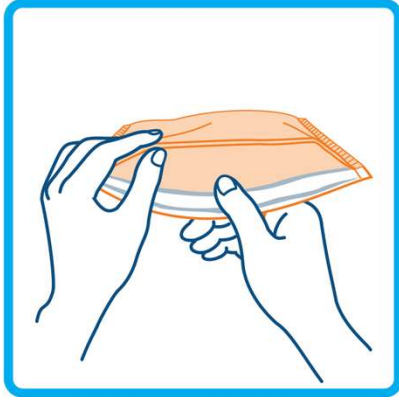


Step 2

Bend the nosepiece to conform mask to your nose.

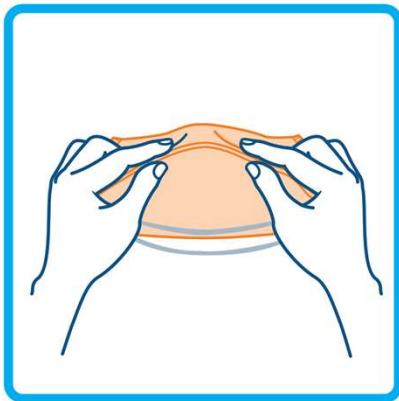

Step 3

Hold the center headbands separately.

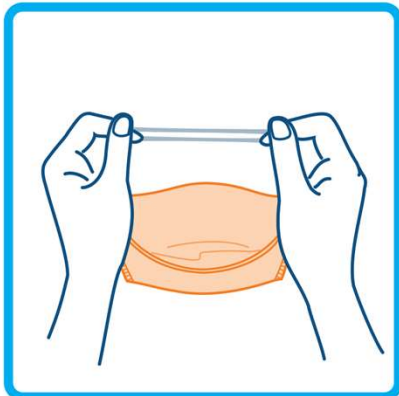

Step 4

Hang the lower part on your chin and bring both bands over your head.



Step 5

Place the upper band at your crown and the lower band at your neck.



Step 6

Form the nosepiece tightly to your face.



To Order:

- Call 800-975-0563
- Scan the QR Code on the right
- Email Info@covenant-hcec.com
- Visit www.covenant-hcec.com

Exclusively Distributed by:

COVENANT
HEALTHCARE EQUIPMENT & CONSULTING



Protection

N95-Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil.
Time use restrictions may apply

Cautions and Limitations

- Not for use in atmospheres containing less than 19.5% oxygen.
- Not for use in atmospheres immediately dangerous to life or health.
- Do not exceed maximum use concentrations established by regulatory standards.
- Failure to properly use and maintain this product could result in injury to health.
- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by manufacturer.
- Refer to user instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- NIOSH does not evaluate respirators for use as surgical masks.

Storage

- Store at room temperature in a clean, dry place, away from direct sunlight.
- Keep the product out of reach of children.



To Order:

- Call 800-975-0563
- Scan the QR Code on the right
- Email Info@covenant-hcec.com
- Visit www.covenant-hcec.com

Exclusively Distributed by:

COVENANT
HEALTHCARE EQUIPMENT & CONSULTING

